#### Instructions for use Contact™ Nipple Shield

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#### EN Contact™ Nipple Shield

#### Intended use

The Medela Contact Nipple Shields are intended to cover and protect the nipples of a mother who is unable to latch on and protect the nipples of a mother who support breastfeeding.

#### Indications

The Contact Nipple Shields are indicated for use when the mother has flat or inverted nipples, sore nipples or suffers from nipple pain. They can also be used when the baby has latch-on problems (such as an atypical sucking technique), when the baby does not transfer the milk well or for preterm infants who fall asleep almost immediately at the breast.

<b>Contraindications</b>
There are no known contraindications.
<b>Important safety information</b>

**Read all instructions before use**

- Use the product only for its intended use as described in this instruction manual.
- Use by more than one person without adequate reprocessing may be a health risk and could cause cross-contamination.
- Use Medela original accessories only.
- Do not use if the packaging is open or damaged before first use.
- Inspect before each use. Throw away at the first signs of damage or weakness.

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**How to use**

Wash your hands thoroughly with drinking-quality water and soap before touching the shield and the breast. Dry your hands with a fresh towel or a single-use paper towel. Clean the breast and surrounding area with a damp cloth. Do not use soap or alcohol which can dry the skin and cause cracked nipples.

Choosing the correct nipple shield size When choosing the correct nipple shield size you need to consider both mother and baby.

Place the shield centrally over the nipple (see figure 3). The open section should point in the direction of the baby’s nose during feeding.

When choosing the correct nipple shield size

**Product variants**

Available in 3 sizes: S/16mm, M/20mm or L/24mm.

Material nipple shield: Silicone  
Material storage box: Polypropylene (PP)

Disposal: According to local regulations

Compliance with the essential requirements of the European Directive 93/42/CEE of 14 June 1993 concerning medical devices.

The packaging contains products intended to come in contact with food.

Do keep the device away from sunlight.

The packaging contains products intended to come in contact with food.

Do keep the device away from sunlight.

Do keep the device away from sunlight.

The packaging contains products intended to come in contact with food.

The packaging contains products intended to come in contact with food.

Do keep the device away from sunlight.

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